

# ENDO-MATE DT2 Standard Sets



## ENDO-MATE DT2 with 20:1 Head

**MODEL**  
ENDO-MATE DT2 (230V)

**ORDER CODE**  
Y1004230

- Control unit
- Motor and Cord
- MP-F20R head (20:1)
- Handpiece stand
- AC adaptor

## ENDO-MATE DT2 without Head

**MODEL**  
ENDO-MATE DT2 W/O (230V)

**ORDER CODE**  
Y1004231

- Control unit
- Motor and Cord
- Handpiece stand
- AC adaptor

### Specifications

- Power supply : AC 230 V 50/60 Hz and Battery pack
- Number of Programs : 9
- Max Torque : 7 Ncm (when using 20:1 head)
- Speed range : 100 -13,000 min<sup>-1</sup> (With 20:1, 1:1 geared heads)
- Charge/Continuous use : 5 h / 2 h (depending on workload)
- Dimensions : W 93 x D 147 x H 100 (mm)
- Weight : Unit 440 g / Motor handpiece and Cord 95 g

\*Files are not included.

### Push type



**MODEL**  
20:1 MP-F20R  
16:1 MP-F16R

**ORDER CODE**  
Y110044  
Y110096

- Miniature Head
- 20:1 / 16:1 Reduction
- For Ni-Ti files (ø2.35)
- Push Button Chuck



16:1 MPA-F16R

**ORDER CODE**  
Y110097

- Miniature Head
- 16:1 Reduction
- For Ni-Ti files (ø2.35)
- Push Button Chuck
- Connection for Apex locator (for long shank)



20:1 MPAS-F20R  
16:1 MPAS-F16R

**ORDER CODE**  
Y110046  
Y110098

- Miniature Head
- 20:1 / 16:1 Reduction
- For Ni-Ti files (ø2.35)
- Push Button Chuck
- Connection for Apex locator (for short shank)

ENDO-MATE DT2 Motor and Cord



MODEL	PRODUCTS	ORDER CODE
ENDO-MATE DT2 Unit		U1138001
ENDO-MATE DT2 Motor and Cord		E1169001
ENDO-MATE DT Adaptor		U480
Handpiece Stand		Z095205

MODEL	PRODUCTS	ORDER CODE
Cord For MPA / MPAS		C876021
Connection Cord Clip		C876509
Rechargeable Batteries		U421070
F-Type Spray Nozzle		Z256090



The heads can be washed in the thermodisinfectior.



The heads are autoclavable up to 135°C.



CREATE IT.

smart  
& portable

ENDO-MATE DT2

ENDODONTIC MICROMOTOR





# ENDO-MATE DT2

ENDO-MATE DT2 has been developed for use with most major brands of Ni-Ti files. The preprogramming function allows highly efficient treatments using different file manufacturer's protocols. The control unit has been designed to be lightweight and compact. It is fitted with a rechargeable battery providing high mobility. The lightweight and slim handpiece allows easy and comfortable handling for high performance.

## Features of ENDO-MATE DT2

1. Compatible all popular brands of Ni-Ti files
2. Torque control, with an auto-reverse function
3. Compact desktop unit
4. 2 way power source (AC Cord / rechargeable battery pack)
5. Lightweight, smart and comfortable handpiece

## Compatible all popular brands of Ni-Ti files

Up to 9 settings (speed and torque etc) can be programmed simply by pushing the memory function button. It's easy to switch between each preset program allowing work with all popular Ni-Ti file manufacturer's protocols.

Gear Ratio	Rotation Speed(min <sup>-1</sup> )
20:1	100~650
16:1	125~800

## Auto reverse function


The auto reverse function activates when the preset torque level has been reached.

## 2 way power source

ENDO-MATE DT2 has been designed to work with a rechargeable battery for portable use or can be powered using the AC adaptor supplied.

## Other function

Beeping sound notifies user when preset torque limit has been reached.

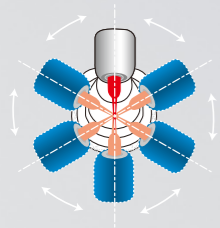

### 6 head angles for ease of treatment

Depending on the position of treatment site, the head angle can be adjusted to the best angle.



### Quick coupling

Quick coupling allows for easy head exchange. All heads are autoclavable up to 135°C.



### 2-way ON/OFF switch

Features are :

- A: Push and release - Micromotor operates continuously until the button is pushed again.
- B: Push and hold - Micromotor operates continuously until the button is released.

### Push type ultra miniature head

Allows fast and easy exchange of files during procedures. Compact head offers excellent visibility of treatment spots.



Lightweight, smart and ergonomic handpiece.

\* actual size